

How to Collect in ProteX



Consult with your fertility team for specific media and lubricant usage instructions.

- 1. Open** your sterile sealed package and place ProteX™ on a flat, stable surface.

If using refrigerated media, ensure it comes to room temperature first.

- 2. Pour** all media from vial carefully into the ProteX.

- 3. Collect** your semen sample into the ProteX container as directed by your fertility team.

If using lubrication, be sure it has been cleared by your fertility team and be careful not to get the lubricant in the ProteX.

- 4. Secure** the lid carefully onto the ProteX container.

- 5. Return** your sample to your fertility team as directed; keep ProteX upright while transporting.

ProteX

**Now give it
your best shot.
Confidently.**

Take it easy, relax at home,
and stay in the moment with
your partner.

You've got this.

-your ProteX team

See reverse for collection instructions.

RSi REPRODUCTIVE
SOLUTIONS